ENGLISH

**OptiBond FL**

**Instructions for Use**

OptiBond FL™ is a two-part primer adhesive system indicated for direct light cured applications. This highly-filled and radiopaque adhesive system provides a true "structural bond" not found in unfilled adhesive systems. With OptiBond FL, adhesion promoters are carried in an ethanol solvent, diminishing the need for multiple coats.

OptiBond FL is available in both bottle and Unidose™ delivery systems.

Directions for using both delivery systems are as follows:

**UNIDOSE Delivery System**

**To dispense OptiBond FL using the Unidose delivery system:**

- Open foil pouch along perforated edge – peel open the pouch.
- Remove the OptiBond FL Unidose yellow (FL Prime) and black (FL Adhesive) capsules and applicator brushes from foil; discard foil.
- Grip Unidose capsule at wings and twist each side in opposite directions until seal is broken completely. Discard side with the long stem (longer half).
- Use applicator brush to apply primer first.
- Then use a new applicator brush to apply adhesive.

**Guidelines for bonding to enamel and/or dentin using the Unidose delivery system:**

1. Isolate prep and dry.
2. Place Kerr Gel Etchant\* with 37.5% phosphoric acid on dentin/enamel for 15 seconds. Rinse with water until etchant has been completely removed – approximately 15 seconds. Gently air dry for a few seconds being careful not to desiccate dentin.

3. Dispense OptiBond FL Prime (Bottle #1) into disposable mixing well. Apply material over enamel and dentin surfaces with a light scrubbing motion for 15 seconds. Gently air dry for approximately 5 seconds. At this point the dentin surface should have a slightly shiny appearance. Do not allow salivary contamination.
4. Dispense and apply OptiBond FL Adhesive (Bottle #2) over enamel and dentin uniformly creating a thin coating. Blow to margin or to thin if necessary using a light application of air.
5. Light cure for 20 seconds\*\*.
6. Proceed with composite placement.

Store at ambient temperature.

**\*CAUTION:** Contains phosphoric acid. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water. Get medical attention for eyes. Do not take internally.

**\*\*Recommended Cure Times when using Kerr Lights:**

|               |              |            |
|---------------|--------------|------------|
| Optilux 501   | Boost Mode   | 10 seconds |
|               | Regular Mode | 20 seconds |
| LEDemetron I  |              | 10 seconds |
| LEDemetron II |              | 5 seconds  |

For all other lights see manufacturer's recommendation.

**CAUTION:** Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

**LIMITED WARRANTY**

**LIMITATION OF KERR'S LIABILITY**

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. Such advice does not expand Kerr's limited warranty or relieve the dentist from controlling applications and methods foreseen by the dentist regarding the intended use of the Kerr Products. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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