Description

AdheSE® One F is a self-etching, light-curing, nano-filled, single-component adhesive for enamel and dentin with fluoride release.

Composition

AdheSE One F contains derivatives of bis-acrylamide, water, alcohol, bis-metacrylamide dihydrogen phosphate, amino acid acrylamide, hydroxyl alkyl methacrylamide, alkyl sulfonic acid acrylamide, highly dispersed silicon dioxide, initiators, stabilizers and potassium fluoride.

Indication

Adhesive for direct light-curing composite and compomer restorations

Contraindication

- Do not use AdheSE One F if a patient is known to be allergic to any of the material's ingredients or if the stipulated working technique cannot be employed.
- AdheSE One F must not be used in combination with self-curing or dual-curing composite materials.
- Do not use AdheSE One F for direct pulp capping.

Side effects

In rare cases, ingredients of AdheSE One F may cause sensitization. In such cases, the product should cease to be used.

Interactions

- Materials containing eugenol or clove oil may inhibit the polymerization of AdheSE One F.
- AdheSE One F cannot be combined with AdheSE DC Activator.

Application

1. A dry operating field must be ensured. Ideally, a rubber dam should be placed (e.g. OptraDam® Plus).
2. In very deep cavities, areas close to the pulp should be selectively coated with a calcium hydroxide liner (e.g. ApexCal®) and subsequently covered with a pressure-resistant cement (glass ionomer cement, e.g. Vivaglass® Liner).
3. Remove the cap of the VivaPen by turning it beyond the point where resistance is felt and simultaneously pulling it off.
4.  Snap on the VivaPen brush cannula. The brush cannula can be rotated up to approx. 270 degrees to the desired position. **Attention:** Do not rotate the cannula beyond the point of resistance; otherwise it will detach!
5. Activate the click mechanism several times to saturate the VivaPen brush cannula with AdheSE One F. The yellow colour of AdheSE One F becomes visible once the brush cannula is saturated.
6. An adequate amount of AdheSE One F is directly applied to the cavity with the VivaPen. Starting with the enamel portion, all cavity surfaces should be thoroughly coated. Activate the click mechanism again if you wish to apply more material. Once the cavity surfaces have been completely coated, the adhesive must be brushed into the entire surface for 20 seconds.
7. Disperse excess amounts of AdheSE One F with a strong stream of air until a glossy, immobile liquid film results.
8. Light-cure AdheSE One F for 10 seconds at a light intensity of more than 500 mW/cm² (with e.g. bluephase® using the LOP mode).
9. Apply the restorative material (e.g. Tetric EvoFlow® or Tetric EvoCeram®) according to the instructions of the manufacturer.
10. Discard the used VivaPen brush cannula after disinfecting the VivaPen with disinfectant wipes and replace it with the original cap. The VivaPen brush cannula is removed by turning it beyond the point of resistance and simultaneously pulling it off. A new VivaPen brush cannula must be attached for every application.

Note

- After each use, the VivaPen must be disinfected using disinfectant wipes for hygienic reasons. Additionally, the VivaPen can be protected against contamination with the VivaPen sleeve.
- Do not use oxidizing disinfectants.
- Do not immerse the VivaPen in disinfectant solutions.
- Do not use other types of brush cannulas.
- The fill-level indicator of the VivaPen identifies the remaining amount of adhesive: If the white indicator is no longer visible on the display, the VivaPen is empty.

Warnings

- AdheSE One F is an irritant. Contact with the skin, mucous membrane or eyes should be avoided. In case of skin contact, immediately wash with copious amounts of water.
- If the material should come into contact with the eyes, immediately rinse with copious amounts of water for at least 15 minutes and consult a physician or ophthalmologist.
- In rare cases, contact with the skin may cause sensitization to the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effects of methacrylamides.

Storage

- Expiry date: see imprint on VivaPen/packaging
- Close VivaPen with the original cap immediately after use
- Storage temperature: 2–28 °C (36–82 °F)

Keep material out of children's reach!

For use in dentistry only!

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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Self-etching dental adhesive with fluoride release

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The Material Safety Data Sheet is available online at www.ivoclarvivadent.com

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.