Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως
Kullanım Kilavuzu

- Skyddande lack som innehåller klorhexidin
- Beskyttelseslak med chlorhexidin
- Klooriheksidiiniä sisältävä suojalakka
- Klorheksidinholdig beskyttelseslakk
- Beschermingslak met chloorhexidine
- Προστατευτικό βερνίκι που περιέχει χλωρεξιδίνη
- Klorheksidin içeren koruyucu vernik

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso

- Protective varnish containing chlorhexidine
- Chlorhexidinhaltiger Schutzlack
- Vernis protecteur avec chlorhexidine
- Lacca protettiva con Clorexidina
- Barniz protector con chlorhexidina
- Verniz protetor com clorhexidina

Instructions for Use

Description

Cervitec Plus is a varnish that protects sensitive dentin in the case of exposed cervicals. Chlorhexidine and thymol are the antimicrobial components in Cervitec Plus. They protect the tooth surface by reducing bacterial activity.

Composition

Ethanol, water, acrylate copolymer, vinyl acetate copolymer and
 Chlorhexidine diacetate 1 %
 Thymol 1 %

Indication

- Protection of exposed root surfaces
- Treatment of hypersensitive cervicals
- Reduction of bacterial activity on tooth surfaces

Contraindication

If patients are known to be allergic to any of the ingredients in Cervitec Plus, the material should not be applied.

Side effects

In individual cases, contact with the mucous membrane may result in short-term, reversible irritation. Chlorhexidine may cause reversible discolouration of the tooth structure and restorations.

Application

Cervitec Plus can be used for patients of all age groups and is professionally applied by dentists, dental hygienists or dental prophylaxis assistants. Generally, Cervitec Plus is applied every three months. If intensive treatment is required, however, the varnish may also be applied more frequently. One Free Stand Single Dose Cervitec Plus is enough for the treatment of one set of teeth. One tube is enough for about 20 to 30 sets of teeth.


CE 0123

604122/0107/WEE/3G

ivoclar
vivadent
clinical

Step-by-step application procedure

1. Clean tooth surfaces thoroughly.
2. Dry with cotton rolls and air syringe.
- 3 a) **Single Dose:**
Remove the foil from the Single Dose and apply the varnish directly from the Single Dose.
- 3 b) **Tube:**
Press out 3 drops into, for example, a dappen dish and reseal the tube.
Hold the tube in a vertical position with the cap facing upwards for opening and closing. For pressing out the contents, hold the opening vertically down.
4. Apply a thin coat of varnish by means of a Vivadent applicator or suitable brush (diagram); in proximal areas, Cervitec Plus is applied with dental floss.
5. Let the varnish dry. The varnish can be dispersed with air.
6. Remove cotton rolls after 30 seconds.
7. Ask patients not to rinse their mouth.

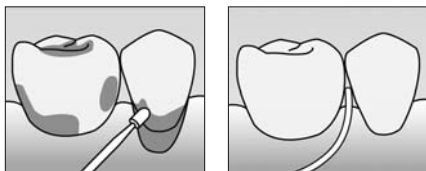Tips for patients

After the application of Cervitec Plus, patients should not eat or drink for 1 hour.

Storage

- Close the tube immediately after use
- Storage at temperature (2-8 °C / 36-46 °F)
- Highly flammable. Keep away from sources of ignition.
- Expiry date: see label / packaging

Keep material out of the reach of children!

For use in dentistry only!

Date information prepared

09/2006 Rev. 0

Manufacturer

Ivoclar Vivadent AG

FL - 9494 Schaan / Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.