Cavifil Injector

English

Instructions for Use	Bruksanvisning
Gebrauchsinformation	Bruksanvisning
Mode d'emploi	Käyttöohjeet
Istruzioni d'uso	Bruksanvisning
Instrucciones de uso	Productinformatie
Instruções de Uso	Οδηγίες Χρήσεως

- Applicator for dental materials in Cavifils
- Applikator für zahnärztliche Materialien in Cavifils
- Pistolet-Applicateur pour matériaux dentaires en Cavifils
- Applicatore per materiali dentali in cavifil
- Aplicador para materiales dentales en Cavifils
- Aplicador para materiais dentários em Cavifils
- Applikator för dentalmaterial i Cavifil
- Applikator til dentalmaterialer i Cavifils
- Applikaattori kapseleissa oleville hammaslääkärin materiaaleille
- Applikator for odontologiske materialer i cavifiler
- Applicator spuit voor tandheelkundig materiaal
- Εργαλείο εφαρμογής οδοντιατρικών υλικών σε μικροσύριγγες Cavifil

For dental use only!

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein, Ivoclar Vivadent AG, FL-9494 Schaan


ivoclar
vivadent®
clinical

Instructions for Use


The Cavifil system consists of the following components:

- Cavifils – unit-dose plastic cartridges (1) filled with composite or other pliable materials
- Cavifil injector (2)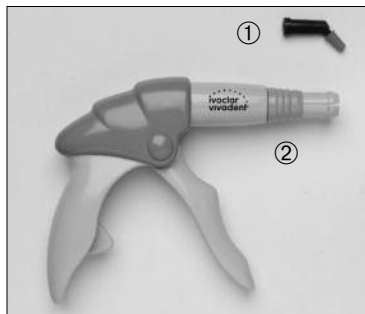

Usage

- Slide the Cavifil into the injector until it snaps into place (3).
- Slide the retaining ring to the very front to ensure that the Cavifil is sufficiently secured. If the retaining ring is not at the very front, there is a risk that the Cavifil will loosen during application.
The Cavifil is now secured in the injector, but it may still be rotated 360° if required (4).
- Activate the lever to extrude the material (5).
The lever cannot be activated if the retaining ring has not been slit to the front to secure the Cavifil in the injector.
- After applying the material, push the retaining ring back and remove the Cavifil from the injector by pulling it out towards the side.

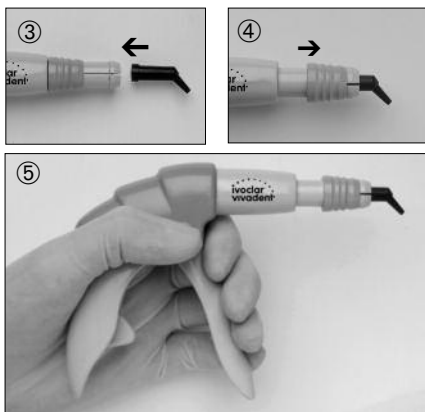Ivoclar Vivadent AG

FL-9494 Schaan/Liechtenstein

This device has been developed solely for use in dentistry. Processing should be done strictly according to the Instructions for Use. Liability cannot be carried out for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the device for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

The direct application of the material into the cavity renders transferring the material from syringe to spatula unnecessary. The cavity can be filled from the bottom without trapping air. There is no excess material to dispose of. The Cavifil system helps to place restorations or sealings economically and efficiently.

The contents of the Cavifils are normally enough for one restoration or four sealings. The Cavifils have been stipulated as a single-dose system. Restorations can be placed economically. In addition, the risk of cross-contamination is minimized.

Notes

- Apply the material at normal room temperature. Material taken from the refrigerator may be difficult to extrude.
- Store Cavifils in a place where they are protected from light.
- The Cavifil Injector can be cleaned with a customary liquid disinfectant. It cannot be autoclaved.

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Ivoclar Vivadent S.A.
Ivoclar Vivadent AB
Ivoclar Vivadent, Inc.