

Adper™ Prompt™ L-Pop™ Self-Etch Adhesive

Preparation:

Clean unprepared tooth structure with a rubber cup and cleaning paste free of fluoride and oil; rinse thoroughly with water spray and dry.

- Squeeze the material from the red reservoir into the yellow (middle) reservoir.
- The middle reservoir will expand indicating proper transfer of fluid.



- Carefully fold back the red reservoir until it meets the flat face of the yellow reservoir.
- **Keep the red reservoir tightly squeezed while bending the package to prevent the material from flowing back.**



- Squeeze the liquid from the yellow reservoir into the green reservoir.



Adper™ Prompt™ L-Pop™ Self-Etch Adhesive



- Apply a churning or spinning motion to the applicator for 5 seconds to mix the adhesive and fully coat the applicator tip.
- A **lightly yellow-tinted applicator tip indicates proper activation.**



- Apply adhesive to the entire surface of the cavity, rubbing in the solution with moderate finger pressure for 15 sec. **Do not shorten this time. Proper bonding will not occur if the solution is simply applied and dispersed.**
- Use a gentle stream of air to **thoroughly** dry the adhesive to a thin film.



- Apply a second coat of adhesive. This coat does not need to be massaged in to the surface.
- Again, use a gentle stream of air to thoroughly dry the adhesive to a thin film.



- Light cure for ten seconds (3 second exposure with a plasma arc lamp).

2 of 2