

Protemp™ Crown Temporization Material




Molar




Bicuspid
(Premolar)




Cuspid
(Canine)

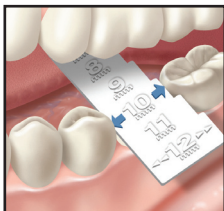

1

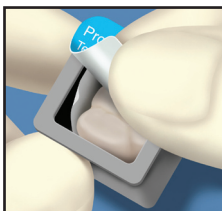


2

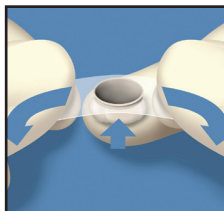


3



4

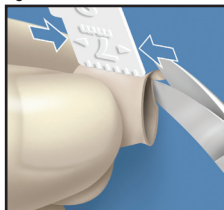


5

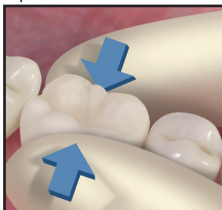

6


Protemp™ Crown Temporization Material

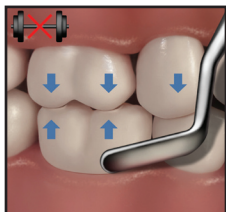


7




8

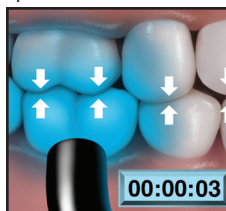


9

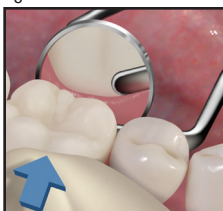


10

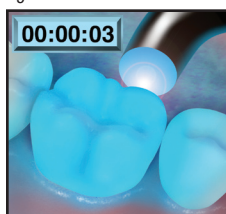


11

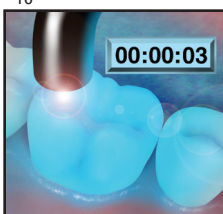

12

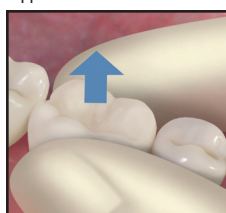


13

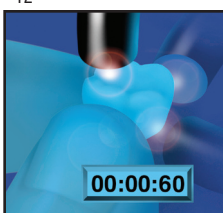


14


Protemp™ Crown Temporization Material

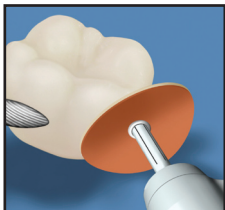


15

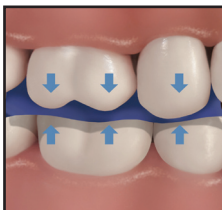


16

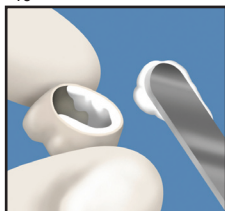

17

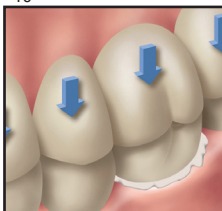


18

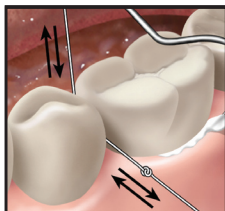

19

Protemp™ Crown

Temporization Material

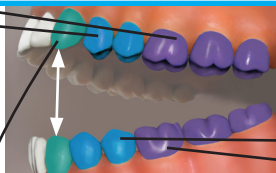

* The wedging point relative to the measuring tool identifies the approximate mesial-distal width and size range for selecting a Protemp Crown.

Size Selection Chart

Crown Name	Approximate Range Mesial Distal Width *
CUSPID (Canine) Universal	
Cuspid (Canine) Universal S	6-8 mm
Cuspid (Canine) Universal L	8-9 mm
BICUSPID (Premolar) Upper	
Bicuspid (Premolar) Upper S	6-7 mm
Bicuspid (Premolar) Upper L	7-8 mm
MOLAR (1st & 2nd) Upper	
Molar Upper S	9-11 mm
Molar Upper L	11-12 mm
MOLAR (1st & 2nd) Lower	
Molar Lower S	9-11 mm
Molar Lower L	11-12 mm
BICUSPID (Premolar) Lower	
Bicuspid (Premolar) Lower	6-9 mm

3M, ESPE and Protemp are trademarks of 3M or 3M ESPE AG.

© 3M 2008. All rights reserved.



3M ESPE

Dental Products

St. Paul, MN 55144-1000



3M ESPE AG

Dental Products

D-82229 Seefeld – Germany



44-0007-4656-8 **4 of 4**