English

DIRECTIONS FOR USE
Alignot™ Alginate Alternative Impression Material
Type 2: Medium Viscosity

General Description
Alignot™ is an outstanding A-silicone impression material intended as an alternative to traditional alginate materials. Quick set time, excellent detail reproduction, compatibility with disinfection products; dimensional accuracy and stability make the best alternative. Available in 1:1 cartridges suitable for the Kerr Extruder and in Volume™ 5:1 380 mL cartridges for use in commercially available mixing machines.

Indications
Alignot™ is suitable for impressions in preparation of:

- Case study models
- Orthodontic models
- Opposing models
- Simple removable dentures
- Removable retainers and splints
- Provisional crown and bridge

Inserting Cartridge into the Extruder

1. Push metal lever at back of extruder up (toward plunger assembly) and pull back on plunger assembly.
2. Lift up on lock assembly on top of extruder exposing slot for insertion of cartridge.

3. Insert cartridge with notch in collar of cartridge down to match protrusion in extruder.
4. Snap lock assembly down locking cartridge into place.

Attaching Mixing Tip to Cartridge

1. While holding cartridge with cap toward you, rotate cap 90° counter clockwise, moving alignment tab on cap to align with alignment notch on cartridge.
2. Separate cap from cartridge by pulling cap off.
3. For first use bleed cartridge by dispensing a small amount of material to ensure even extrusion from each side of the cartridge. Wipe excess material from cartridge.
4. Take mixing tip and align alignment tab with notch on cartridge.
5. Push mixing tip down onto cartridge. Rotate cartridge 90° clockwise. This will lock mixing tip on cartridge.

Extruder Operation

1. Squeeze trigger with moderate and even pressure to begin mixing material.
2. Material will stop flowing when trigger is released.

Volume™ Cartridge Assembly

1. Place the cartridge on a flat surface with the nozzles pointing up
2. While securing the cartridge with one hand, lift the tab covering the nozzles with the opposite hand until it forms a 90 degree angle with the nozzles
3. Remove the tab from the nozzles
4. Place the cartridge into the mixing machine
5. Make sure to bleed the material from the cartridge to ensure an adequate mix of base and catalyst material.
6. Affix a Kerr Volume mix tip to the cartridge, verifying that the hex portion of the mix tip is fully engaged with the mixing rod on the automatic mixing machine.
7. Place mix tip collar over tip and turn clockwise with finger pressure until collar feels secure.

General Timelines: Work time is based on material at 23°C (74°F). Set or removal time is based on material at 37°C (99°F). These times may vary based on storage conditions, temperature, humidity, and age of product.

Alignot Cartridge:	
Working Time:	1 minute from start of mixing.
Removal Time:	2 minutes and 30 seconds from start of mixing.
Alignot FS Cartridge:	
Working Time:	1 minute from start of mixing.
Removal Time:	2 minutes from start of mixing.
Alignot Volume™	
Working time:	1 minute and 15 seconds from start of mixing.
Removal time:	2 minutes and 45 seconds from start of mixing.
Alignot FS Volume	
Working time:	1 minute and 15 seconds from start of mixing.
Removal time:	2 minutes and 15 seconds from start of mixing.

General Impression Taking Guidelines

1. Select an appropriate impression tray for the procedure.
2. Always use Kerr VPS adhesive with impression trays.
3. Always bleed cartridge (extruding a small amount of material from each side of the cartridge) and wipe cartridge tip before installing a new mixing tip.
4. Use a moderate even pressure to extrude material through the mixing tip.
5. Leave the mixing tip in place after each use as it acts as a seal.
6. Maintain a dry field for optimum impressions.
7. Check to ensure materials are set prior to mouth removal.

Cautions

- Work time is based on material at 23°C (74°F). Set time is based on material at 37°C (99°F). These times may vary based on storage conditions, temperature, humidity, and age of product.
- Use of products containing active sulfur, aluminum chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials, latex gloves, etc) in conjunction with this product will interfere with the setting reaction of the polyvinylsiloxane material. Use of these materials requires the area to be rinsed thoroughly to remove all residues.
- Do not use material after expiration date.

Model Pouring
Models may be poured immediately after removal. Pouring of models may be delayed up to 14 days without loss of accuracy.

Disinfection of Impressions
Impressions may be safely disinfected in 2 % glutaraldehyde solutions. Refer to manufacturer's directions for time, dilution and solution temperature recommendations.

Storage
Store at ambient temperature. Avoid prolonged exposure to heat.

LIMITED WARRANTY
LIMITATION OF KERR'S LIABILITY
Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product.
In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.
EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages. EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.