English

Vinylpolysiloxane Impression Material System

- Type 3: low consistency - light bodied
- Type 2: medium consistency - medium bodied
- Type 1: high consistency - heavy bodied
- Type 0: very high consistency - putty

General Description

Extrude is a system of vinylpolysiloxane impression material viscosities suitable for all crown and bridge, edentulous and implant impressions. Extrude is offered in four viscosities to address the complexity of each case.

Inserting Cartridge into Extruder Gun

(Note: Extruder gun comes fully assembled and requires no assembly by the user)

- Push metal lever at back of extruder up (toward plunger assembly) and pull back on plunger assembly until assembly is fully retracted.
- Lift up on lock assembly on top of extruder exposing slot for insertion of cartridge.
- Insert cartridge with notch in collar of cartridge down to match protrusion of extruder.
- Snap lock assembly down locking cartridge into place.

Attaching Mixing Tip to Cartridge

- While holding cartridge with cap toward you, rotate cap 90° counter clockwise, moving alignment tab on cap to align with notch on cartridge.
- Separate cap from cartridge by pulling cap off.

- For first use bleed cartridge by dispensing a small amount of material to ensure even extrusion from each side of the cartridge. Wipe excess material from cartridge.
- Take the mixing tip and align alignment tab with notch on cartridge. Push mixing tip down onto cartridge. Rotate 90° clockwise to lock mixing tip onto cartridge.

Attaching Intra-Oral Tip to Mixing Tip

- Insert the round end of the intra-oral tip into mixing tip.
- Press intra-oral tip into mixing tip. A click should be heard when the intra-oral tip locks into place.

Extruder Operation

- Squeeze Extruder trigger with moderate and even pressure to begin mixing material. Material will stop flowing after trigger is released.
- For subsequent usage, remove the used mixing tip and squeeze the Extruder trigger to extrude material from the two openings of cartridge. Wipe end of cartridge. Install a new mixing tip and twist one quarter turn to lock in position.
- To remove the cartridge, hold the release lever up and withdraw plunger in a backward direction.

Putty Instructions

- Brush a thin coat of Kerr VPS tray adhesive on the selected tray. Allow a minimum of 10 minutes to air dry for maximum adhesion.
- Measure equal amounts of Extrude XP Putty base and catalyst.
- Mix base and catalyst with fingertips until a homogenous color is achieved (approx. 30 seconds).
- Place the mixed putty into the adhesive coated tray.
- Your technique will determine the need for a putty spacer prior to insertion of the putty filled tray into the mouth. Once inserted into the mouth, immobilize the putty without applying excessive pressure until the materials are completely set.
- Remove the tray from the mouth by applying downward pressure along the edge of the tray.

Cleaning

- This Extruder may be cleaned with commercially available detergents.
- Do not use an organic solvent that may tend to dissolve the plastic dispenser and plunger.
- The Extruder may also be cold sterilized in an approved commercially available solution. Consult manufacturer's recommendation for dilution, time and bath temperature directions for the specific solution involved.
- The Extruder may be autoclaved at 270°F (132°C) and 30 PSI for 30 minutes.

General Timelines (23±2 °C)		
	Total Working Time* from start of mix	Minimum Removal Time from start of mix
Extrude Wash	3:00 minutes	6:00 minutes
Extrude Medium	3:00 minutes	6:00 minutes
Extrude Extra	3:00 minutes	6:00 minutes
Extrude MPV	2:15 minutes	5:15 minutes
Extrude XP Putty	2:00 minutes	6:00 minutes
* ± 0:15 minute		

General Impression Taking Guidelines

- Always use Kerr VPS Adhesive with impression trays. Allow a minimum of 10 minutes to air dry for maximum adhesion.
- Always bleed cartridge (extruding a small amount of material) and wipe cartridge tip before installing a new mixing tip.
- Leave the mixing tip in place after each use or reattach cartridge cap.
- Maintain a dry field for optimum impressions.
- Keep tip of intra-oral tip in material when syringing around the preparation.
- Check to ensure materials are set prior to removal from the mouth. Minimum intra-oral time is 3:00 minutes for Extrude Wash, Medium, Extra and MPV, and 4:00 minutes for Extrude XP Putty.
- For putty dispense equal volumes of putty base and putty catalyst using the color coded scoops. Do not interchange scoops.
- Knead the putty base and catalyst together with fingertips until a homogenous colored mix is obtained. The mixing should be completed in 30 seconds.

Cautions

- Handling retraction cord with latex gloves may prevent setting of the impression material in the area of glove contact with the retraction cord.
- Always bleed cartridge after removing old mixing tip or cap. Wipe cartridge tip prior to installing the new mixing tip.
- If die stone is hand mixed, delay pouring the impression for 1 hour to obtain the best die stone surface.
- Do not use material after expiration date.
- Kerr VPS Adhesive contains solvent, use in a well ventilated area. Avoid contact with skin or eyes, wear protective gloves and eye protection when using adhesive.
- Use of products containing active sulfur. Aluminum chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfid impression materials,latex gloves , etc.) in conjunction with this product will interfere with the setting reaction of the vinylpolysiloxane material.
- Mixing Extrude XP Putty with latex gloves will inhibit the set. Wash hands thoroughly with soap and water before handling putty.
- Do not use the same putty scoop for both base and catalyst as it will cross contaminate the product.

Model Pouring

The impression may be poured 20 minutes after start of mix if a vacuum mixed die stone is used. If hand mixed die stone is used, delay pouring of impression for 1 hour.

Pouring of impression may be delayed up to 14 days without loss of accuracy.

Storage

Store at ambient temperature.

Shelf Life

Refer to outer package for expiration date. Do not use after expiration date.

Disinfection of Extrude Impressions

Extrude impressions may be safely disinfected in 2% glutaraldehyde solutions. Refer to manufacturer's directions for time, dilution and solution temperature recommendations.

Electro Plating of Extrude Impressions

Extrude impression may be electro plated. Recommended plating solutions are as follows:

Silver Plating Solution	
Silver cyanide	21.43%
Potassium cyanide	57.14%
Potassium carbonate	21.43%

Copper Plating Solution	
Anhydrous copper sulfate	222-250 gm.
Concentrated H2SO4	75 ml.
Phenol	10 ml. or gm.
Ethanol	25-50 ml.
Distilled H2O	1.0 liter

Limited Warranty - Limitation of Kerr's Liability

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EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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