Instructions For Use

Maxcem Elite™

self-etch / self-adhesive resin cement

TABLE OF CONTENTS

ENGLISH	4
FRANÇAIS	9
ESPAÑOL	15
PORTUGUÊS	21
TIẾNG VIỆT	27
中文体	32
中文版	37
日本語	42
한국어	50
ภาษาไทย	56
KERR WARRANTY	62

ENGLISH

Maxcem Elite™ Self-etch/Self-adhesive Resin Cement

Description

Maxcem Elite is a self-etch, self-adhesive resin cement offered in a paste/paste formulation indicated for indirect cementation of porcelain, ceramic, resin, and metal restorations. This product contains 69% filler by weight and is radiopaque. The dual-cure material is packaged in dual-barrel syringes with single-use automix tips and optional curved dispensing tips to allow user to deliver desired volume of cement directly into restoration and/or tooth prep.

Indications for Use

Indirect restorations

Cementation of porcelain, ceramic, resin, and metal-based inlays/onlays, crowns, bridges, posts, and veneers

Cementation of crown restorations to implants

Instructions for Use

Note: Verify that the laboratory has pre-treated the final restoration in accordance with manufacturers instructions. Automix tips are for single patient use only. Use only temporary cements that are eugenol-free. The use of eugenol-containing temporary luting materials can inhibit the polymerization process of resin cements including Maxcem Elite.

All Indirect Restorations (excluding Veneers)

1. Remove provisional restoration.

2. Use OptiClean rotary instrument, or your choice of cleaning method, to remove temporary cement and residue from preparation area. Rinse thoroughly.
3. Air dry preparation (do not desiccate).
4. Try in restoration to ensure proper fit.
5. Select desired shade of cement.
6. Bleed cartridge before initial use. Place appropriate mixer on dual syringe cartridge. Mixer with wide opening is only for use with curved tips.

Dispensing of the material can be done as follows:

Inlay/Onlay Dispense the cement directly into cavity preparation covering all surfaces. Seat restoration gently onto tooth prep, allowing the cement to flow from all sides.

Crown Dispense the cement directly into restoration or tooth prep, covering all surfaces. Seat restoration gently onto tooth prep, allowing the cement to flow from all margins.

Post Prepare post space. Size and fit post. Dispense the cement onto post or directly into canal space. Seat post and vibrate slightly to avoid trapped air. Allow the cement to slowly flow from canal space.

7. When restoration is properly seated, remove excess cement. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack-curing excess with a light for approximately 2 seconds, or by allowing the cement to self-cure for approximately 2-3 minutes after application.
8. After removal of excess cement, light cure all surfaces including margins using Demi curing light for 10 seconds. See footnote for recommended curing times for other curing light models.*

9. Use finishing strips for proximal areas. Polish margins with Gloss Plus discs, cups, or points. Use HiLusterPlus polishers for a final high luster shine.
10. Check occlusion.

Veneer Restorations

For cementation of veneers (ideal for 1-2 units), select OptiBond Solo Plus adhesive or OptiBond All-In-One adhesive. The following instructions are for OptiBond Solo Plus adhesive only:

1. Etch tooth prep for 15 seconds with Kerr Gel Etchant (37.5% phosphoric acid), rinse thoroughly, and air dry (Do not desiccate).
2. Apply OptiBond Solo Plus adhesive with light brushing motion for 15 seconds, air thin for 3 seconds, and light cure with Demi curing light for 5 seconds.*
3. Dispense cement directly onto internal surface of the veneer. Place the veneer gently onto tooth, allowing the cement to slowly flow from all margins.
4. Spot cure the veneer into place on facial surface, away from the margins, using a small diameter light guide for several seconds.
5. After removing excess cement, light cure all surfaces using Demi curing light for 10 seconds.*

For veneer cementation using Maxcem Elite, adhesive application on tooth prep is required. Using an adhesive in combination with Maxcem Elite for other cementation procedures is optional but not necessary.

Crown Restorations to Implants

1. Remove provisional restoration.
2. Use OptiClean rotary instrument, or your choice of cleaning method, to remove temporary cement and residue from the implant.

3. Air-abrade the implant or roughen the implant surface with BluWhite Diamond Bur, or your choice of bur.
4. Rinse the implant thoroughly.
5. Air dry the implant.
6. Try in restoration to ensure proper fit.
7. Select desired shade of cement.
8. Bleed cartridge before initial use. Place appropriate mixer on dual syringe cartridge. Mixer with wide opening is only for use with curved tips.
9. Dispense the cement directly into crown restoration or onto implant, covering all surfaces. Seat restoration gently onto the implant, allowing the cement to flow from all margins.
10. When restoration is properly seated, remove excess cement. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack curing excess with a light for approximately 2 seconds, or by allowing the cement to self-cure for approximately 2-3 minutes after application.
11. After removal of excess cement, light cure all surfaces including margins using Demi curing light for 10 seconds.*
12. Use finishing strips for proximal areas. Polish margins with Gloss Plus discs, cups, or points. Use HiLuster Plus Dia polishers for a final high-luster shine.
13. Check occlusion.

Note: Material minimum working time is 2 minutes at ambient temperature; Set time is approximately 3 minutes at oral temperature. These times may vary based on storage conditions, temperature, humidity, and age of product.

*** Recommended Cure Times:**

Demi & L.E.Demetron II: 5 seconds for adhesives & 10 seconds for cements. L.E.Demetron I & Optilux 501: 10 seconds for adhesives & 20 seconds for cements. For all other lights, see manufacturer's recommendation.

STORAGE: Store at ambient temperature. Avoid excess heat. For optimum freshness, refrigeration is optional.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact. Kerr Gel Etchant contains Phosphoric Acid. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water. Get medical attention for eyes. Do not take internally.

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Limited Warranty

Limitation of Kerr's Liability

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

SYBRON DENTAL SPECIALTIES

Manufactured by:

Kerr Corporation

1717 West Collins Avenue
Orange, CA 92867 USA
(800) KERR-123
kerrdental.com

Distribution Facilities:

Kerr Australia Pty. Ltd.

Unit 11, 112-118 Talavera Road
North Ryde 2113
New South Wales, Australia
+61-29-870-7500

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Japan, Inc.**