	English
CLASSIFICATION	
TempBond™	ISO 3107 Type I, Class 1 - Zinc Oxide/Eugenol Cement
TempBond™ NE	ISO 3107 Type I, Class 1 - Zinc Oxide/Non-Eugenol Cement
TempBond™ Clear	Dual Curable Transparent Resin Cement with Triolosan

DESCRIPTION

TempBond temporary cement product series consists of TempBond, TempBond NE and TempBond Clear. Each is designed to be suitable for various applications such as cementing temporary crowns, bridges, inlays, onlays and splints. It has excellent flow to permit the restoration to be easily and completely seated. It is strong enough to withstand the stresses of mastication, yet permits easy removal of the restoration when this is desired. TempBond and TempBond NE are available in traditional tubes, Unidose™ single-use foil pouches, and automix dual barrel syringes. TempBond Clear is available in an automix dual barrel syringe, and contains triclosan.

DIRECTIONS FOR USE

Work Time and Set Time:

At the ideal mixing conditions of 70-77 °F (21-25 °C) and 40-60% relative humidity, the following work time and setting time from start of mixing are obtained:

	Work Time (mm:ss)	Set Time (mm:ss)
TempBond	≥ 1:30	≤ 7:00
TempBond NE	≥ 1:30	≤ 7:00
TempBond Clear	≥ 1:30	≤ 5:30

Mixing:

Unidose: Extrude equal lengths of Base and Accelerator onto the mixing pad provided. The length to be extruded will depend on the size and type of restoration to be cemented. Replace caps tightly on tubes after use. Thoroughly mix the pastes for approximately 30 seconds.

Unidose: Cut along dotted line with scissors, dispense contents of pouch onto mixing pad. Thoroughly mix the pastes for approximately 30 seconds. Syringe: Remove cap from syringe. Always bleed syringe upon the initial use. Place automix tip onto syringe. Turn automix tip 90 degree to lock in place. The material is now ready to be dispensed directly onto the temporary or the tooth prep. No hand mixing is necessary.

Preparation and application:

Dry the prepared teeth and the surface of the restoration. Apply a thin layer of the mixed cement to the internal surfaces of the temporary restoration. Firmly seat the restoration in the mouth. After the material has set remove excess material with a scaler or other instrument.

NOTE

- A softer cement can be obtained by adding up to 25%, by volume, white petrolatum in TempBond and TempBond NE.
- To accelerate the set TempBond Clear may be cured with a visible light for 20 seconds per surface.

CAUTIONS

- TempBond contains eugenol. A small number of patients may experience an allergic reaction to eugenol. Caution is recommended with these patients.
- TempBond NE may contain trace amounts of eugenol.
- TempBond Clear contains (meth)acrylic resins. Uncured (meth)acrylic resins may cause contact dermatitis and damage the pulp. Avoid prolonged contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

STORAGE

Store at ambient temperature.

SHELF LIFE

Refer to outer package for expiration date. Do not use after expiry date.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. Such advice does not expand Kerr's limited warranty or relieve the dentist from controlling applications and methods foreseen by the dentist regarding the intended use of the Kerr Products. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

	Dansk
KLASSIFICERING	
TempBond™	ISO 3107 Type I, klasse 1 – zinkilte/eugenol cement
TempBond™ NE	ISO 3107 Type I, klasse 1 – zinkilte/eugenol-fri cement
TempBond™ Clear	Dualhærdende transparent plasticement med Triolosan

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