Ti-Core® / Ti-Core® Natural Instructions

Ti-Core and Ti-Core Natural are hybrid filled autocuring composites, reinforced with titanium (or Ti-Core Natural with lathanide). The addition of titanium or lathanide increased the composite's compressive strength comparable to that of dentin. For the first time, similar compressive strengths give the dentist the ability to construct a core which mimics the natural strength of dentin, enabling the dentist to create a superior and more fatigue resistant substructure.

AREA OF USE:

1. **NONVITAL TEETH** - Ti-Core is recommended for use with all prefabricated posts and/or pin build-ups.
2. **VITAL TEETH** - Ti-Core may be used with pins to build-up missing tooth structure. A calcium hydroxide or glass ionomer base may be used under the Ti-Core composite.

BONDING TO ENAMEL & DENTIN:

- a. **Ti-Core can be bonded to enamel with the acid etch technique.**
- b. **Ti-Core can be bonded to dentin using any of the available dentin bonding agents currently on the market.**

MIXING TI-CORE:

1. Using the rounded end of the disposable plastic spatula, remove from the jar of CATALYST PASTE an amount slightly less than the volume of the cavity to be filled. Place on mixing pad.
2. Using the flat, squared end of the disposable plastic spatula, remove from the jar of BASE PASTE an amount approximately equal to the Catalyst Paste placed on the mixing pad in step 1.
3. Place Base Paste next to Catalyst Paste on mixing pad. **REPLACE CAPS ON JARS.**

4. Using the flat, squared end of the disposable plastic spatula, mix the two pastes for about 20 seconds using 1 to 2 strokes per second. The objective is to make the mixture as homogeneous as possible.
5. **To make Ti-Core set faster, add more base. To make Ti-Core set slower, add more catalyst.**

PLACEMENT:

1. To create an intimate adaptation between root, post and core material, place Ti-Core into a core form and seat over the root and post with firm pressure.

2. Alternatively, you can place a matrix band around the prepared tooth and post, syringe Ti-Core and apply light pressure for an intimate fit.

TOOTH PREPARATION:

After six minutes from the beginning of mixing the core build-up material, remove the core form or matrix band. Diamond or carbide burs may be used to prepare the tooth. Air and water cooling is recommended to cut the material more efficiently.

	MIX TIME	PLACEMENT TIME	SETTING TIME (DO NOT DISTURB)	HARDENING TIME	REMOVE MATRIX FINISH AND POLISH
	20 SEC.	1 MIN. 40 SEC.	20 SEC.	2 MIN.	
0			2	4	6

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