OptraSculpt

Instructions for Use Gebrauchsinformation Mode d'emploi Istruzioni d'uso Instrucciones de uso Instruções de Uso Bruksanvisning Brugsanvisning Käyttöohjeet Bruksanvisning Productinformatie Οδηγίες Χρήσεως

- Composite Sculpting & Modelling instrument
- Modellier-Instrument für Composites
- Instrument à sculpter et à modeler les composites
- Strumento di modellazione per compositi
- Instrumento para el Modelado de composites
- Instrumento para modelar e esculpir compósitos
- Modelleringinstrument för komposit
- Modelleringinstrument til komposit
- Yhdistelmämuovien muotoiluinstrumentti
- Modelleringinstrument for kompositter
- Modelleerinstrument voor composieten
- Εργαλείο διαμόρφωσης συνθέτων ρητινών

For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

590853/0704/WE3/G

ivoclar
vivadent®
clinical

English

Instructions for Use

Description

The OptraSculpt system consists of two instruments and disposable elastic modeling tips in six different shapes. OptraSculpt allows easy application and manipulation of all composite materials due to the low adhesion of the modelling tips to composites.

Shapes

Shapes of the handles

Modelling tips

Recommended applications:

1 Cylinder

Initial increments of Class I & II restorations

2 Chisel

Marginal ridges, cervical margins of Class V fillings

3 Spatula

Class V fillings, all facial and buccal surfaces (or smooth surfaces in general). Also recommended for the removal of excess cement

4 Ball

Developing concavity of the lingual surface on anterior restorations.

5 Point

Creating detailed anatomy such as pits and fissures, lingual contours, refining margins.

6 Pyramid

Creating detailed anatomy such as pits, fissures, and other detailed morphology.

Indication

Sculpting and contouring anatomy in direct placement, composite resin restorations.

Use

Choose and attach the appropriate modeling tip to the instrument.


If you do not dispense the material directly from the capsule, use the end with the metal spatula to place the material into the preparation. Use the elastic modeling tips to adapt, contour and sculpt the material.

The different shapes can be used as follows:

Cylinder

The cylinder-shaped modeling tips are used for placing and levelling the first increment in Class I and II cavities. They provide optimum adaptation to the tooth structure.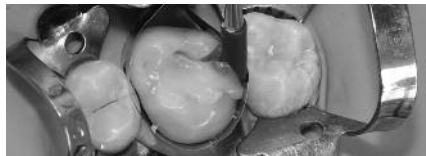

Chisel

The chisel-shaped modeling tips can be used for Class V restorations, labial and buccal surfaces as well as occlusal surfaces.

Spatula

Spatula-shaped modeling tips are particularly suitable for contouring the labial surfaces of anterior restorations (Class III and IV) and Class V restorations.

Ball

Ball-shaped modeling tips can be used to develop concavities on the lingual surface of anterior restorations. (Class III and IV).

Point

The pointed modeling tips have been designed for contouring and sculpting the occlusal anatomy (Class I and II) as well as the lingual surfaces of anterior restorations (Class III and IV).

Pyramid

It is recommended to use the pyramid-shaped tips to accentuate fissures (Class I and II) and to create detailed morphology.

Notes

- The OptraSculpt handle can be autoclaved.
- The modelling tips are intended for single use and must be removed prior to sterilization in an autoclave.
- As small parts present a potential aspiration hazard, the use of rubber dam is recommended.

**Keep out of the reach of children!
For use in dentistry only!**

**Date information prepared: 05/2004
Revision 0**

Manufacturer:
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.
Dentigenix Inc.