Instructions for Use

Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως
Kullanım Kılavuzu
**Инструкция по
применению**
Instrukcja stosowania

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Bruksanvisning
Bugsanvisning

- Joustava apuväline helpottamaan toimenpiteisiin pääsyä suussa
- Flexibel hjelpemiddel for fri tilgang til munnhulen
- Flexibel hulpmiddel voor vrije toegang tot de mondholte
- Εύκαμπτο βοηθήμα για εύληρη πρόσβαση στο στόμα
- Ağiz ortamına girişi kolaylaştıran esnek yardımcıdır
- эластичное вспомогательное средство для свободного доступа к полости рта пациента
- Elastyczne narzędzie pomocnicze, ułatwiające dostęp do pola operacyjnego podczas zabiegów stomatologicznych

- Flexible aid for full mouth access
- Flexibles Hilfsmittel für freien Zugang zum Mundraum
- Ecarteur flexible permettant un accès facile en bouche
- Retrattore flessibile per l'accesso al cavo orale
- Ayuda flexible para el acceso completo a la cavidad bucal
- Instrumento flexível para acesso completo da área bucal
- Flexibelt hjälpmedel för fritt arbetsfält
- Flexibelt hjælpemiddel til bedre adgang til mundhulen

Description

OptraGate® ExtraSoft is an auxiliary aid for clinical use which allows the working field to be accessed with ease. It provides retraction of lips and cheeks during dental treatment, offers the clinician increased visibility and accessibility and facilitates moisture control in the oral cavity. The device, which features high three-dimensional flexibility and elasticity, ensures patient comfort and assists in keeping the patient's mouth open. OptraGate is available in sizes Regular, Small as well as Junior for children between 5 and 10 years of age. The modified "ExtraSoft Version" offers a snug, particularly gentle fit in the area of the upper and lower lip frenulum which minimizes intra-oral pressure.

Composition

Thermoplastic elastomer (SEBS) and polypropylene.
OptraGate ExtraSoft is latex-free.

Indication

Suitable for use in diagnostic, preventive and therapeutic dental procedures (in particular restorative procedures, cavity preparation, professional tooth cleaning, diagnostic procedures, scaling & root planing, tooth whitening, impression making).

Contraindications

- The use of OptraGate ExtraSoft is contraindicated
- in oral surgical and implantological procedures
 - if the patient is known to be allergic to any of the components of OptraGate ExtraSoft

Description of the procedure

The elastic material of OptraGate ExtraSoft is stretched by mean of two rings. The thicker ring of the device is positioned in the area of the gingivobuccal fold, while the thinner ring, which features two tabs, remains outside the mouth. The elastic component between the two rings thus embraces the lips of the patient and provides retraction due to the restoring force of the rings (see Figure 1).

For dental use only.

Made in Switzerland
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

59074/1010/WE3/G

Fig 1: OptraGate ExtraSoft in place with the tabs pointing downward. It does not interfere with the patient's occlusion.

1. Select the appropriate sized OptraGate ExtraSoft. The size (R for regular, S for small or J for Junior) is printed on the package as well as on the left tab of OptraGate ExtraSoft.
Note: It is advisable to moisten OptraGate ExtraSoft with water prior to using it, as this will facilitate insertion.
2. OptraGate ExtraSoft may be inserted from both the 12 o'clock position with the patient lying down, or from the 8 o'clock position with the patient sitting upright (or, the 4 o'clock position if the clinician is left-handed).
3. To ease placement, with the tabs pointing downward, the operator should grasp the thicker intraoral ring by reaching through the thinner extraoral ring. The inner ring should be held between the thumb and middle finger and pressed together slightly (see Figure 2).

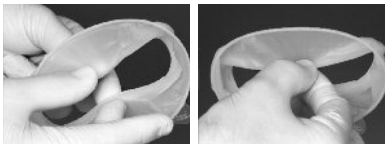
Fig 2: Correct hold on the intraoral ring with thumb and middle finger. The intraoral ring is slightly compressed in the process.

4. The slightly compressed intraoral ring is then inserted into the buccal corridor on one side, so that the elastic component embraces the corner of the mouth and the outer ring is still located extraorally (see Figure 3).

Fig 3: The thicker intraoral ring is positioned between the teeth and the corner of the mouth.

5. As soon as the OptraGate ExtraSoft is securely in place on one side, the other side is inserted in the same manner by slightly bending it (see Figure 4).

Fig 4: The intraoral ring is positioned behind the two corners of the mouth.

6. Subsequently, the intraoral ring is placed behind the lower and upper lips and thus OptraGate ExtraSoft assumes its final, stable position (see Figure 5). The intraoral ring can be inserted with more ease if the patient's mouth is relaxed.

Fig 5: Final positioning behind the lower and upper lip.

If, in individual cases, the intraoral ring should show a tendency to slip out of the gingivobuccal fold if the mouth is closed completely, positioning of the intraoral ring deeper inside the vestibule is usually sufficient to solve the problem. Sometimes, the use of another size may also be a solution.

The securely positioned OptraGate ExtraSoft improves the overall view and accessibility of the working field. Lateral movements of the mandible and opening of the mouth to various degrees may provide additional space and access to the treatment area. OptraGate ExtraSoft may even be left in place when checking occlusion.

7. To remove OptraGate ExtraSoft, grasp the extraoral ring in the lower area with the help of a paper towel and lift it slightly so that the intraoral ring is loosened from the lower gingivobuccal fold. (Figure 6). The intraoral ring can then be easily removed from the upper gingivobuccal fold and hygienically disposed of using the paper towel.

Fig 6: Removal of OptraGate ExtraSoft

Hinweis

-  OptraGate ExtraSoft is not sterile and thus not suitable for use with oral surgical or implantological procedures.
-  OptraGate ExtraSoft is for single use only.
- It cannot be disinfected or sterilized. The material is not suitable for sterilization. Disinfecting agents may have a detrimental effect on the material and lead to undesirable reactions in the patient.
- When used in tooth whitening procedures, care should be taken that no bleaching material gets caught between OptraGate ExtraSoft and the oral soft tissues to prevent an irritation of the skin and mucous membrane.

Keep out of the reach of children!
For use in dentistry only!

Date of preparation: 09/2010, Rev. 3

Manufacturer:

Ivoclar Vivadent AG, 9494 Schaan,
Principality of Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.