

**Diapaste™** is a root canal filling material for pulp capping and an infected root canal. It is also applied a good intracanal medicament. It contains Calcium Hydroxide and Barium Sulfate mainly, having good radiopacity and antibacterial functions.

**Diapaste™** is very stable without any solidification or separation and it has a highly-flowable premixed paste in a convenient syringe.

Also **Diapaste™** has excellent accessibility to the apex and biocompatibility to the periapical tissue.

## Compositions

- Calcium Hydroxide
- Barium sulfate
- Propylene glycol
- Thickening agent
- Water

## Indications

- Treatment for an infected root canal
- Temporary root filling
- Root canal filling for primary teeth
- After pulp extirpation treatment
- Filling on pulpotomy
- Temporary pulp capping

## Recommended Procedure

1. After root canal preparation, rinse and dry the canal gently.
2. Insert the tip of the syringe into the root canal.
3. Apply paste by compressing the syringe piston gently and smoothly. The indicator on the syringe is noticeable.
4. Withdraw the syringe gently when the paste fills the canal completely.
5. Remove excess paste on the surface.
6. Remove the disposable tip of syringe, close the cap tightly and store in a cool place.

## Packages

**Diapaste™** Type A (ITEM # 1001-401)  
2g prefilled syringe 1ea. , 20 disposable tips

**Diapaste™** Type B (ITEM # 1001-402)  
2g prefilled syringe 1ea. , 10 disposable tips

**Diapaste™** Type C (ITEM # 1001-403)  
2g prefilled syringe 1ea. , 4 disposable tips


**Diapaste™** Refill Kit (ITEM # 1001-404)  
2g prefilled syringe 1ea.

**Diapaste™** Intro Kit (ITEM # 1001-405)  
0.5g prefilled syringe 1ea., 4 disposable tips

## Precautions

- **Diapaste™** is used by a dentist and dental professional for dental purpose only and instruction for use showed be carefully read prior to using the **Diapaste™**
- Do not use **Diapaste™** to a patient having inflammation or allergy.
- Do not press paste with excess force, the patient may have some acute pain.
- In accidental contact with eyes occurs, flush immediately with a large amount of water.
- Avoid exposure to direct sunlight.
- Store in a cool place (Temperature: 10°C ~ 25°C)
- Please keep the syringe cap tightly when not using **Diapaste™** for a long time.
- Do not use as an internal medicine.
- Do not use after expiry date.

## Symbols

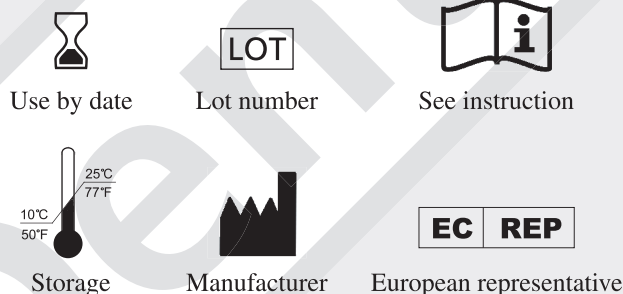

## Warranty

DiaDent Group International will replace product that is proven to be defective. DiaDent Group International does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

## Contact Information

### DiaDent Group International

P.O Box 35 Cheongju si 360-600 Choongchong Buk-do, Korea  
Tel : 82-43-266-2315 Fax : 82-43-262-8658  
<http://www.diadent.co.kr> E-mail : [diadent@chollan.net](mailto:diadent@chollan.net)

### DiaDent Group International Inc.

11-3871 North Fraser Way, Burnaby British Columbia, Canada, V5J 5G6  
Tel : 1-604-451-8851 Fax : 1-604-451-8865  
<http://www.diadent.com> E-mail : [diadent@diadent.com](mailto:diadent@diadent.com)

### DiaDent Europe B.V.

Antennestraat 70, 1322AS Almere, the Netherlands  
Tel : 31-36-549-8607 Fax : 31-36-536-7317  
<http://www.diadent europe.com> E-mail : [diadent@diadent europe.com](mailto:diadent@diadent europe.com)

## European Representative

DiaDent Europe B.V.  
Antennestraat 70, 1322AS Almere,  
the Netherlands  
Tel: 31-36-549-8607 Fax: 31-36-536-7317